

Retractable Scalpel



About Us


PL Medical Co., LLC is pleased to offer a retractable scalpel to meet the needs of safety-conscious medical professions.

The innovative PL Medical Co., LLC retractable scalpel fits hands comfortably and is demanded in plastic and reconstructive surgery, neurosurgery, orthopedic surgery and oral surgery.

The PL Medical Co., LLC single use scalpel is designed to provide utmost safety to health care providers from sharp edge related injuries. The scalpel blade shield is molded with an anti-slip grip surface and an easy to operate push button lock. For the safe disposal of used scalpels, the rigid plastic protective blade shield is easily engaged by moving it forward past the visible white safety line on the handle, locking it in place to prevent contact with the blade after use.

Advantages

- Safe Disposal - You can be confident that once the protective shield has been locked into place, it can be disposed of without presenting a potential hazard for accidental contact with the blade
- Comfortable to use - The PL Medical Co., LLC safe, ergonomic design feels just like regular single-use scalpels and doesn't require any change in handling technique while in use
- Easy to view scale - The PL Medical Co., LLC Safe handle has an embossed, easy to read 5cm measurement scale.

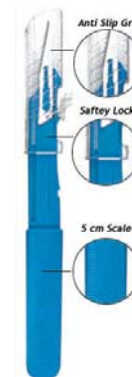

Applications

- Hospitals
- Primary Care
- EMS
- Pediatric
- Universities and National Laboratories

Usage

1. The retractable scalpel is packaged in the safety position. To use the scalpel, simply press down on the locking button and move the protective shield back until you hear a click. It is now ready for use.
2. To cover the blade during a procedure, press down on the locking button and move the protective shield forward until you hear a click.
3. Finally, when the procedure is complete, press down on the locking button again and move the protective shield forward past the safety line visible on the handle, and the blade is now safely protected for final disposal.

Product Description

- Available in sterile sizes 10, 11 and 15

Sizes Offered:

Sterile:

Item#	SPECS	PKG	Price
DSR10	#10	Box/10	CALL
DSR11	#11	Box/10	CALL
DSR15	#15	Box/10	CALL

PL Medical Co., LLC
321 Ellis Street
New Britain, CT 06051

T: (800) 874-0120
T: (860) 223-8882
F: (860) 223-5941

info@plmedical.com
orders@plmedical.com